

Thermal Angel®

Blood and IV Fluid Infusion Warmer

TA-200 Quick Steps



Step 1:

- Remove the female luer cap from the inlet end of the Thermal Angel.
- Connect IV tubing to the female luer. Tighten until resistance is met. Do not over tighten!

Step 2:

- Remove the male luer cap from the outlet end of the Thermal Angel.
- Connect 9 Inch IV Extension Set to the male luer. Tighten until resistance is met. Do not over tighten!

Step 3:

- Connect assembled IV line to fluid source and prime the line.

Step 4:

- Connect the male end of the Power Cable to the inlet end cap of the Thermal Angel.
- Connect the female end of the Power Cable to the Battery.
- The green LED on the Thermal Angel inlet end cap should display continuous illumination without blinking.

Step 5:

- Connect 9 Inch IV Extension Set to IV catheter.
- Engage roller clamp on IV tubing to adjust flow.

Estill Medical Technologies, Inc.

4144 N. Central Expwy. Suite 260 Dallas, Texas 75204

(214) 561-6001 Fax: (214) 561-1930 Toll Free: (877) 354-0286 www.thermalangel.com